

# RESTORIGIN<sup>™</sup> Sx

A Natural Next-Generation Tissue Barrier

## Quick Reference Guide

### Instructions for Use

1. Remove the pouch containing the Restorigin Sx allograft from the box packaging.
2. Utilizing aseptic technique, peel open the outer peel pouch from the chevron end and present the inner pouch to the operative field (if required).
3. Wait to open the inner pouch until ready to place the Restorigin Sx allograft.  
(NOTE: Locate the Orientation Indicator Sticker for proper placement of the Restorigin Sx allograft.)
4. When ready, locate the tear notch on the inner pouch and tear open.
5. Grasp the Restorigin Sx allograft and place it directly on the surgical wound site.  
(NOTE: Locate the Triangular Notch for proper placement of the Restorigin Sx allograft.)

### Restorigin<sup>™</sup> Sx Orientation Diagram

- Proper orientation for Restorigin Sx is dependent on the procedure, based on physician usage. For surgical wound coverings, the epithelial side should be placed toward the wound bed.
- Epithelial side up orientation for Restorigin Sx occurs when the allograft is held with the triangular notch positioned in the upper left hand corner of the allograft (as shown below) OR the Orientation Indicator Sticker on the inner pouch is facing upwards.

Epithelial side up



Epithelial side down



## Restorigin™ Sx Amniotic Patch, Thin

Product Number	Description	Size
RSX-AM-0101	Restorigin Amnion Patch, Thin	1x1cm
RSX-AM-0202	Restorigin Amnion Patch, Thin	2x2cm
RSX-AM-0203	Restorigin Amnion Patch, Thin	2x3cm
RSX-AM-0212	Restorigin Amnion Patch, Thin	2x12cm
RSX-AM-0303	Restorigin Amnion Patch, Thin	3x3cm
RSX-AM-0404	Restorigin Amnion Patch, Thin	4x4cm
RSX-AM-0406	Restorigin Amnion Patch, Thin	4x6cm
RSX-AM-0408	Restorigin Amnion Patch, Thin	4x8cm
RSX-AM-0707	Restorigin Amnion Patch, Thin	7x7cm
RSX-AM-1010	Restorigin Amnion Patch, Thin	10x10cm
RSX-AM-1012	Restorigin Amnion Patch, Thin	10x12cm

## Restorigin™ Sx Amniotic Patch, Medium

Product Number	Description	Size
RSX-AC-0101	Restorigin Amnion Patch, Medium	1x1cm
RSX-AC-0202	Restorigin Amnion Patch, Medium	2x2cm
RSX-AC-0203	Restorigin Amnion Patch, Medium	2x3cm
RSX-AC-0404	Restorigin Amnion Patch, Medium	4x4cm
RSX-AC-0406	Restorigin Amnion Patch, Medium	4x6cm
RSX-AC-0408	Restorigin Amnion Patch, Medium	4x8cm



Parametrics Medical has used reasonable efforts to provide accurate and complete information herein, but this information should not be construed as providing clinical advice, dictating reimbursement policy, or as a substitute for the judgment of a health care provider. It is the health care provider's responsibility to determine the appropriate treatment, codes, charges for services, and use of modifiers for services rendered and to submit coverage or reimbursement-related documentation.